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Instruction for use LAPAROX[®] Cryo cylinders

Composition, presentation:

- Carbon dioxide (CO₂), 100%
- LAPAROX Cryo is packaged as liquefied gas under pressure in gas cylinders. 1 liter of LAPAROX Cryo corresponds to 0,75 kg at a temperature of 15°C.
- The gas cylinders are equipped with dip tubes allowing liquid phase being discharged.

Intended application:

- LAPAROX Cryo is intended to be used as an accessory to a CE-marked medical device for administration of carbon dioxide.
- LAPAROX Cryo is used in cryotherapy as cooling/freezing media for topical procedures.

Special warnings/possible side-effects:

- This product is exclusively intended for cryotherapy. Do not use it for any other application.
- LAPAROX Cryo should always be administered in accordance with the instructions from the medical device manufacturer, by educated personnel.
- CO₂ (a gas heavier than air) can accumulate on low points and make the atmosphere dangerous. Good ventilation is required in areas where LAPAROX Cryo is used.
- LAPAROX Cryo is discharged from the cylinder in liquid phase. Do not open a cylinder with dip tube without prior connection (risk of cold burns by liquid potential projection).

Precautions for use:

- Contains (liquified) gas under pressure; may explode if heated.
- Handle material with clean hands.
- Do not transfer pressurized gas from one cylinder to another.
- Do not stand in front of the valve outlet or behind the pressurized gas cylinder, but always at the side.
- Use standard and specific connection fittings or hoses for medical CO₂.
- Unless explicitly specified, use the cylinder in an upright position.
- The use of the product at high flow can cause icing at the cylinder and connections.
- Do not attempt to repair a defective valve.
- Do not disconnect or try to tighten the connection without first depressurizing the system.

Instruction for use:

- Before use, refer to the manufacturer's instructions of the device associated with the LAPAROX Cryo and follow the recommendations of the healthcare professional.
- At delivery and prior to any use, check the package integrity and the tamper evidence sealing system.
- In particular, check whether the equipment is suitable and in good condition. Ensure that the residual amount will allow continuity of supply throughout the product delivery process by weighing the cylinder. Since LAPAROX Cryo is a liquid gas under pressure, the pressure drops quickly as soon as the cylinder no longer contains any liquid.
- If necessary, clean and decontaminate the cylinder and check its integrity.
- Remove the seal (tamper evidence) from the package, check that the valve is clean and that no foreign particles are present.
- Open the cylinder valve slightly until the gas flows out and immediately close the valve again (this will remove any particles). Repeat this step 2 to 3 times.
- Connect the cylinder to the corresponding device:
 - Screw connection: align the regulator and cylinder connections.
 - Pin index connection: align the locating pins of the valve to the holes.

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(052020/pdf)

- Secure the connection by hand until it reaches the stop.
- Before opening the valve check that the couplings are properly fitted.
- **Do not open the cylinder valve if not connected (risk of cold burns).**
- Slowly open the cylinder valve completely, without forcing.
- Suddenly opening the valve may lead to liquifying the product and could pose a risk of cryogenic burning.
- After use, always close the cylinder valve and depressurize the system.

Special recommendations:

- For medical use by qualified personnel. Use should be by competent and qualified personnel according to the instructions for use with the administration device.
- When transporting the cylinders of LAPAROX Cryo, secure the cylinders to avoid any risk of shocks or falling.
- Handle cylinders with caution and particularly those with a capacity of < 5 liters (no protective cap on Pin-Index taps).
- Do not lift the cylinder by its valve. Keep empty cylinders with valve closed.

Storage/transport:

- Handle carefully.
- Do not exceed the expiry date which is indicated on the package label.
- Keep in well-ventilated areas, away from sources of heat, bad weather and at temperatures below 50°C.
- Storage areas must be sufficiently ventilated.
- In storage, during transport and in use: secure the cylinders with an appropriate means (chains, hooks...) to keep them in a vertical position and avoid any risk of accidental falling over.
- Store and transport upright with closed valve and pressure regulator removed.
- Not MRI compatible.

Possible incidents:

- Gas leak: close the valve, purge the coupling system. If the leak persists, purge the system from the cylinder, allow the gas to be evacuated without attempting to seal or repair the valve and return the defective packaging to the manufacturer.
- Any adverse event must be reported to the national competent authority in accordance with the national vigilance system for medical devices:
For The Netherlands: IGJ - Inspectie Gezondheidszorg en Jeugd:
 E-mail: meldpunt@igj.nl.
For Belgium: FAGG - Federaal agentschap voor geneesmiddelen en gezondheidsproducten: E-mail: meddev@fagg.be.
For Luxembourg: Ministère de la Santé, Direction de la Santé:
 E-mail: meddevices@ms.etat.lu.

Further Information:

- This instruction for use was last updated: March 2020.
- Please refer for further information relating to cryogenic gases and Liquid Carbon Dioxide to the MSDS available from www.linde-healthcare.nl/shop or contact the manufacturer (see below).

Manufacturer: Linde Gas Therapeutics Benelux B.V.
 De Keten 7, 5651 GJ Eindhoven, The Netherlands
 The Netherlands: Tel. +31 40 2825825
 Belgium & Luxembourg: Tel. +32 70 233826

(052020/pdf)